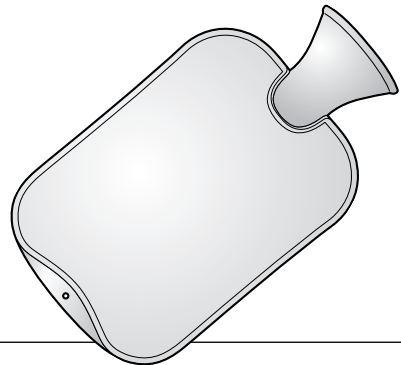
Wärmflasche
Hot-water bottle
Bouillotte
Borsa per l'acqua calda
Termofor
Melegvizes palack
Sıcak su torbası

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Liebe Kundin, lieber Kunde!

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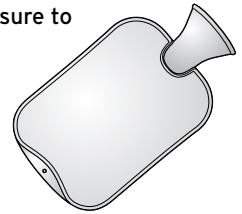
366 658 / Stand: 2018-07

Dear Customer

This leaflet contains important information regarding the use of this product. Please read these instructions carefully to ensure effective, long-lasting use of this product. Keep these instructions for future reference. If you pass this product on to someone else, be sure to give them these instructions.

Your Tchibo Team

Intended use

The hot-water bottle is designed to be filled with hot water and is used for warming the human body.

Usage

The hot-water bottle is suitable for ...

- ... stomach pain (such as digestive problems, menstrual pains;
not for inflammations such as appendicitis, etc.),
- ... muscular pain (such as sore and aching muscles),
- ... muscular tension,
- ... cold hands and feet (caused by low blood pressure).

Do **not** use the hot-water bottle if you have...

- ... inflammations,
- ... circulatory disorders,
- ... feverish illnesses,
- ... reduced nerve sensitivity / inability to sense heat,
- ... extensive skin injuries or skin diseases,
- ... chronic venous insufficiency.

If you use this product regularly and repeatedly on one area of skin, it may cause the skin disease erythema ab igne to appear. This will heal up once you stop the heat treatment. The hot-water bottle must only be used for its intended purpose.

For your safety

This product must not be used by anyone who is unable to perceive heat properly. Small children are more sensitive to heat. Certain disorders, such as diabetes, may be accompanied by impaired temperature perception. Disabled or frail persons may not be able to call for help if they become too warm. Therefore, never use this product to treat helpless or sick persons or those insensitive to heat, as well as children under the age of 3, as children under 36 months have much more sensitive skin. Heat should not be used to treat inflammation. When in doubt, consult your doctor or pharmacist before use.

To prevent burns, test the temperature with the back of your hand before use.

Caution: Please note that temperatures that exceed 42 °C can cause burns on sensitive skin.

Do not fall asleep with the hot-water bottle, as any skin burns may go unnoticed at first.

Warning! Not a children's toy. Store the hot water bottle and all parts out of reach of children. Only use under adult supervision.

Filling with water

- Fill the hot-water bottle with water that is **hot, but not boiling**.
We recommend a water temperature of max. 60 °C.
- Never fill the bottle more than two-thirds full.
- Hold the hot-water bottle by the neck in an upright position and fill it slowly to avoid hot water splashing.
- Do not overfill the hot-water bottle as this could cause it to burst.
- Expell excess air from the hot-water bottle by slowly lowering it onto a flat, level surface until water appears at the opening. **Proceed with caution to avoid scalding yourself!**
- Do not fill with water taken directly from the water supply as this could shorten the life of the hot-water bottle.
- Firmly screw the stopper in tight to prevent leaking. Never use tools to do this - screwing the stopper in hand-tight is sufficient! Only use the original stopper.
- Before every use, make sure that the hot-water bottle is tightly sealed and that the filler is empty.

Use

WARNING! The hot-water bottle can cause burns. Avoid prolonged or direct skin contact.

Only use the bottle with its cover or on top of clothing. Please note that temperatures exceeding 42 °C can cause burns on sensitive skin! Never use the hot-water bottle as a pillow. Do not sit on the hot-water bottle.

Never heat the hot-water bottle in an oven, grill or microwave! Do not add anything else to the water, e.g. salt, essential oils etc.

Storage and care

- Empty the hot-water bottle completely after use.
-  Store the bottle with the stopper removed in a cool, dry and dark place. Protect it from exposure to direct sunlight. Do not place anything on top of the bottle. Protect the bottle from contact with hot surfaces, fire, acids, oils and grease, etc. Keep sharp and pointed objects away from the hot-water bottle as these could damage it.
- Clean the hot-water bottle under running water without any cleaning products.
- The cover is machine-washable. Please follow the care instructions on the sewn-in label. After washing, stretch the cover back into its original shape and let it dry completely before using it again.

Tip

The stopper and edge of the hot-water bottle each have a connecting hole. These holes can be used to attach the stopper to the hot-water bottle so that it does not get lost.

Durability

As with any other material, this hot-water bottle is subject to normal wear and tear. It will last for several years depending on environmental factors and frequency of use. Check the bottle and stopper for signs of wear or damage before every use. If you notice any damage, replace the hot-water bottle with a new one - we recommend replacing it after 5 years at the latest. This product can be disposed of with household waste.

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2018-07

This product complies with the
European Directive for Medical
Products 93/42/EEC.

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Hugo Frosch GmbH